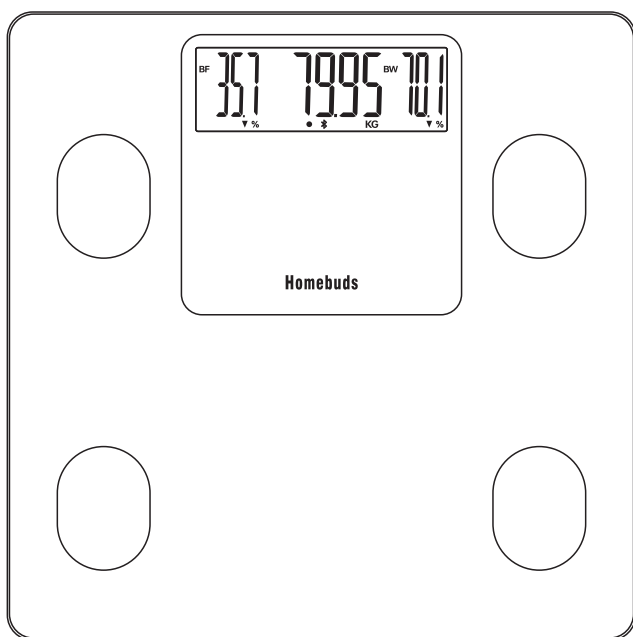
User Manual

Smart Body Fat Scale

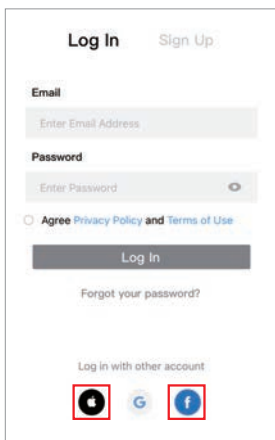
Model:HB9123

Please read this User Manual carefully before using the device


IOS QUICK START GUIDE


1. Search "Vitafit" via Apple Store.

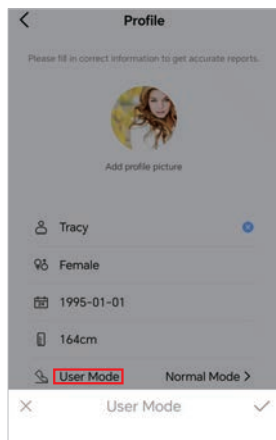
2. Convenient login by Apple or Facebook account.

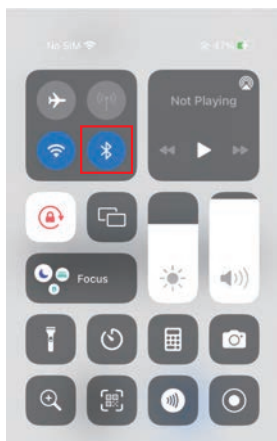
3. Choose suitable Mode. Note: Weight Only Mode specialized for pregnant or having a pacemaker, etc.

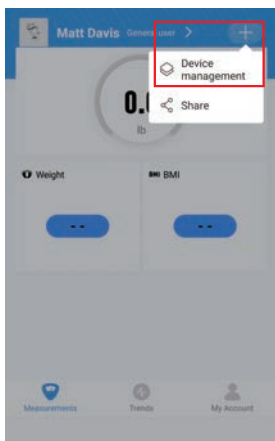4. Turn on Bluetooth on your phone.

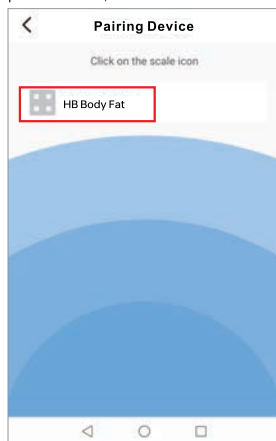5. Pairing with Homebuds Scale.

Notes: 1. For body fat measurement, make sure your feet bare and dry when measuring.

2. For excellent experience, recommend to reassemble batteries for the first beginning use.

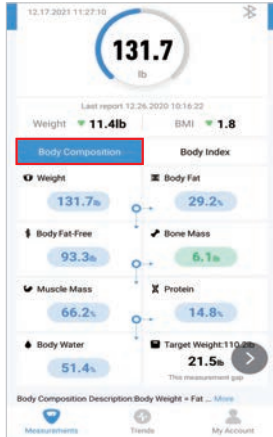
3. UI will be different depended on scale type: Smart Weight, Smart Body Fat.

4. Notes: When weight range between from 10kg/22lb-200kg/440lb, body composition such as body fat could be scientifically analyzed; When weight range LESS THAN 10kg/22lb OR MORE THAN 200kg/440lb, ONLY WEIGHT COULD BE MEASURED.



IOS QUICK START GUIDE



6. Turn on Vitafit App and step on for measuring.

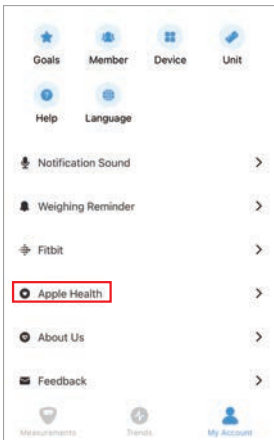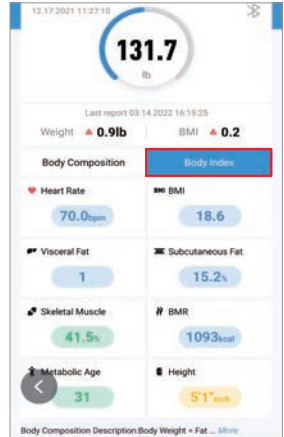7. Measurement data.

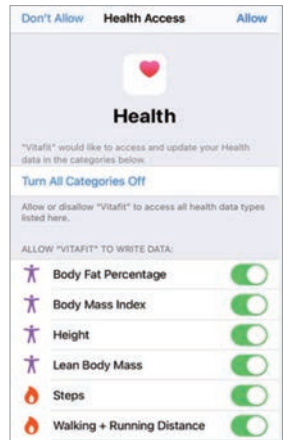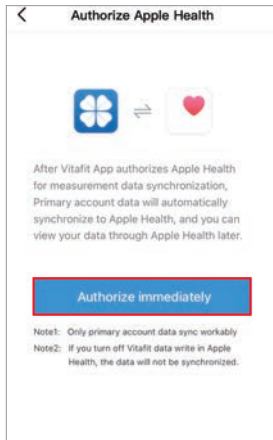
8. Set up for sync with Apple Health.


9. View measurement data in Apple Health after sync.

- Notes: 1. For body fat measurement, make sure your feet bare and dry when measuring.
2. For excellent experience, recommend to reassemble batteries for the first beginning use.
3. UI will be different depended on scale type: Smart Weight, Smart Body Fat.
4. Notes: When weight range between from 10kg/22lb-200kg/440lb, body composition such as body fat could be scientifically analyzed; When weight range LESS THAN 10kg/22lb OR MORE THAN 200kg/440lb, ONLY WEIGHT COULD BE MEASURED.

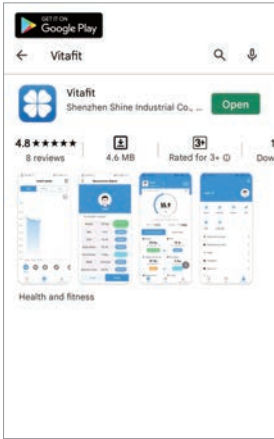
ANDROID QUICK START GUIDE

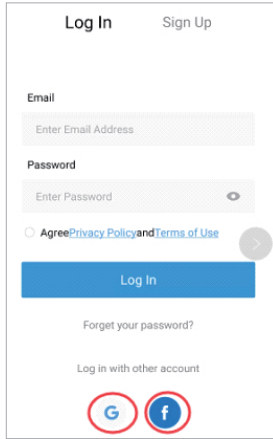
1. Search "Vitafit" via Google Play Store.

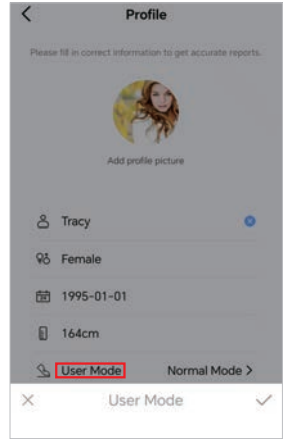
2. Convenient login by Google or Facebook account.

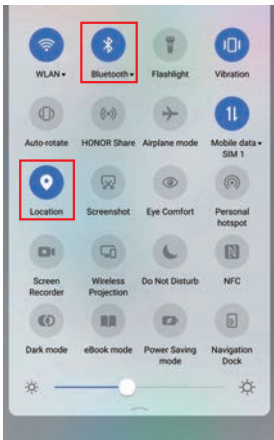
3. Choose suitable Mode. Note: Weight Only Mode specialized for pregnant or having a pacemaker, etc.

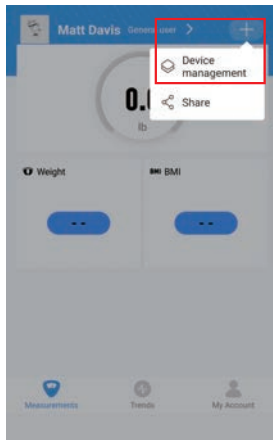4. Turn on Bluetooth and Location on your phone, **OR ELSE THE APP DON'T GET RESULTS ACCORDING TO ANDROID REGULATIONS.**

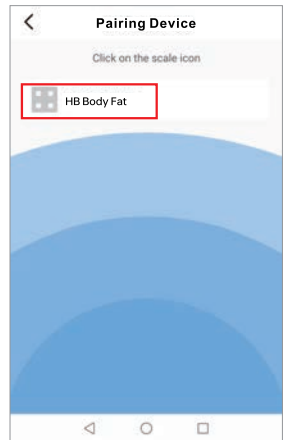5. Pairing with Homebuds Scale.


Notes: 1. For body fat measurement, make sure your feet bare and dry when measuring.
2. For excellent experience, recommend to reassemble batteries for the first beginning use.
3. UI will be different depended on scale type: Smart Weight, Smart Body Fat.
4. Notes: When weight range between from 10kg/22lb-200kg/440lb, body composition such as body fat could be scientifically analyzed; When weight range LESS THAN 10kg/22lb OR MORE THAN 200kg/440lb, ONLY WEIGHT COULD BE MEASURED.



ANDROID QUICK START GUIDE


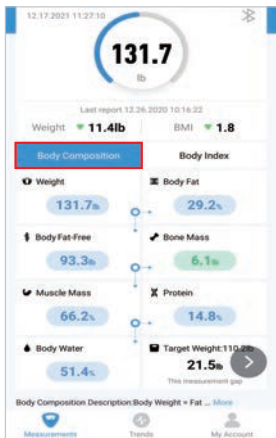
6. Turn on VitaFit App and step on for measuring.

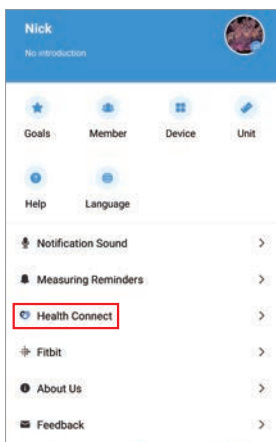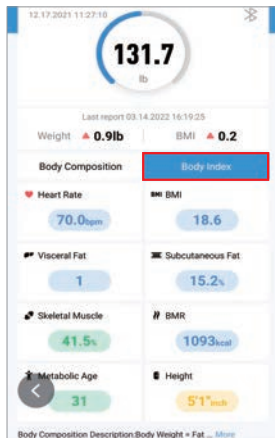7. Measurement data.

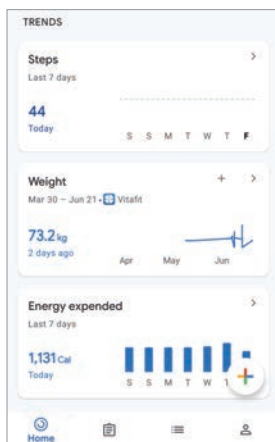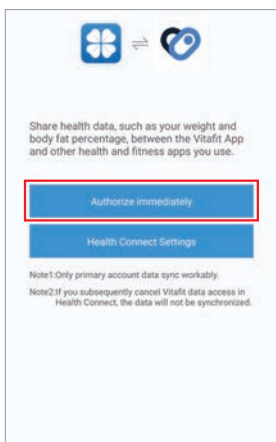8. Set up for sync with Google Fit.

9. View measurement data in Google Fit after sync.

- Notes: 1. For body fat measurement, make sure your feet bare and dry when measuring.
2. For excellent experience, recommend to reassemble batteries for the first beginning use.
3. UI will be different depended on scale type: Smart Weight, Smart Body Fat.
4. Notes: When weight range between from 10kg/22lb-200kg/440lb, body composition such as body fat could be scientifically analyzed; When weight range LESS THAN 10kg/22lb OR MORE THAN 200kg/440lb, ONLY WEIGHT COULD BE MEASURED.

CONTENTS

01-07 English

08-14 Deutsch

15-21 Français

22-28 Español

29-35 Italiano

Safety Cautions

1. This scale is not intended for commercial use.
2. This scale should not be used to diagnose or treat any medical condition. Consult your primary care physician before making changes to your diet, exercise plans, or physical activities.
3. **If you are pregnant, have a pacemaker, or other internal device, you should not use the body fat function of this smart scale. Please note that all data should be used as a reference and is not substitute for medical advice from a licensed healthcare professional.**
4. Check the scale before use. Do not use the scale if it is damaged in any way.
5. The glass platform is slippery when wet. Make sure the platform and your feet are dry before stepping on for measurement.

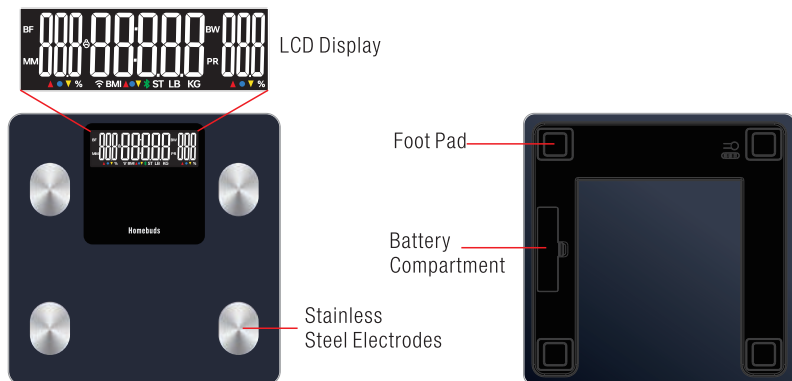
Use and Care

1. To weigh safely and accurately, stand with your feet shoulder-width apart while on the scale.
2. DO NOT strike or drop the scale.
3. DO NOT stand on the edge of the scale or jump on it.
4. DO NOT stand on the scale platform for a prolonged period of time or when not in operation.
5. DO NOT store or operate near heat sources or in humid environments.
6. Handle the scale with care, as it is a precision instrument.
7. Clean the scale with a clean and dry cloth. Avoid getting water or any other liquid inside the scale.
8. DO NOT use chemical or abrasive cleaning agents to clean the scale.
9. Keep the scale in a cool and dry environment.
10. Always keep the scale in a horizontal position, do not store vertically.
11. If the scale fails to turn on, check whether the batteries are installed correctly. Replace the batteries if necessary.
12. If an error appears on the display or if the scale does not turn off automatically, remove the batteries for over 30 seconds, then reinstall them.
13. DO NOT overload the scale (maximum weight capacity: 250kg/550lb/39st).
14. DO NOT mix new and used batteries or different battery brands to power the scale.

Product Overview

Thank you for choosing Homebuds Smart body fat Scale. This scale is your personal health assistant. It uses bio-electrical impedance analysis (BIA) technology to offer 13 composition data you need to keep track of your personal health: Body Fat, Body Fat-free, Bone Mass, Muscle Mass, Protein, Body Water, BMI and much more. We sincerely hope that you enjoy using your new Homebuds scale.

IMPORTANT NOTICE: When weight range between from 10kg/22lb-200kg/440lb, body composition such as body fat could be scientifically analyzed; When weight range LESS THAN 10kg/22lb OR MORE THAN 200kg/440lb, ONLY WEIGHT COULD BE MEASURED.

Cal	Calibrate	Calibrate the scale before each use	
Err	Overloaded	(Max.250kg/550lb/39st)	
Lo	Low Battery	Replace the batteries timely	
00000	Measuring Body Fat		
Bluetooth icon	Bluetooth	Bluetooth icon lights up means connected	
ST LB KG	Units	Switch via the app when connected; Switch via scale when not connected	
▲ ● ▼	▲ Extremely High ● Standard ▼ Low		
BMI	BMI	Body Mass Index	
BW	Body Water(%)	BF	Body fat(%)
PR	Protein(%)	MM	Muscle Mass(%)

Weighing with our App

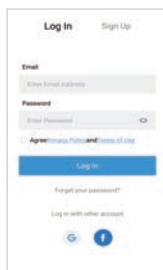
The Homebuds Bluetooth Scale can connect with your mobile phone or Ipad via Bluetooth . The Body Weight and body fat result data will be showed at the App named "Vitafit", which is easy to download from IOS App Store or Google Play.

Download App and Register ID

Scan the QR code or Download " Vitafit" via IOS App Store or Google Play.


Register Your own ID,Then add personal data and confirm OK.

Special User Mode Setting

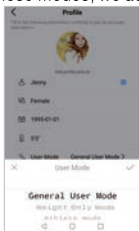Under these modes, we adopt zero-current technology that only measure weight and BMI.

1. If you are pregnant, have a pacemaker or other internal device, please choose Weight Only Mode for safety measurement.

2. If the user is under 2 years old, put in baby's age and height, Vitafit app will automatically recognize and enter baby mode that only measure weight.

3. If the user is among 2-18 years old, put in child's age and height, Vitafit app will automatically recognize and enter child mode.

Note: When measuring with Vitafit App, please switch the unit via App, the scale will keep same unit with App. Let's make more clear, the unit button on the backside of scale will be nonfunctional when the scale connected to Vitafit App.

Pairing the Homebuds Scale with your smartphone

Turn on Bluetooth® on your smartphone.

NOTE: For Android users, Location and Bluetooth® must be turned on for Android 6.0 and later.

OR ELSE THE APP DON'T GET RESULTS ACCORDING TO ANDROID REGULATIONS.

1. Remove the insulator strip from battery compartment
2. Place the scale on a hard flat floor surface.
3. Open Vitafit app, and tap "+" on the top right corner of the "Device Management" page to pair.
4. Step on the scale with bare feet to turn on scale, then the device name will be appeared on the app.
5. Add the device named "HB Body Fat" to connect. Once pairing is successful, the Bluetooth icon will turn solid on the display.

MEASURING

IMPORTANT: If the scale not connected with Vitafit app, the scale only display weight result.

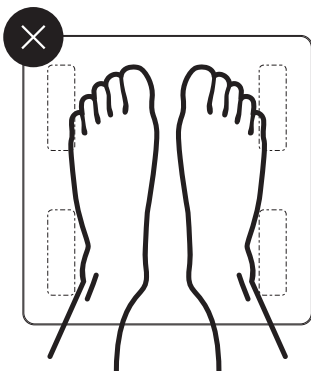
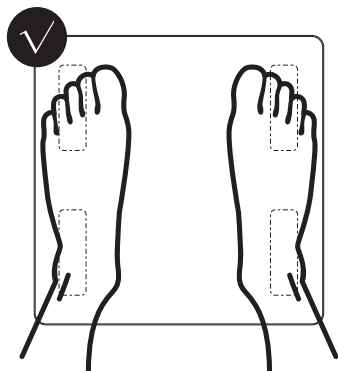
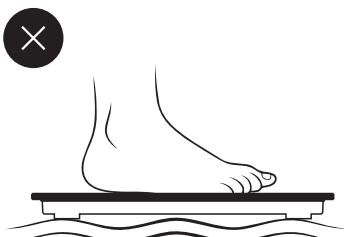
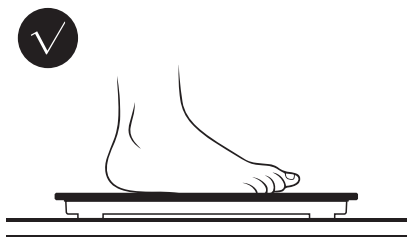
To directly obtain the other data of BMI, body fat, body water, muscle mass and protein from the scale's display, you must connect the scale to Vitafit App and create an account first.

If the weight is 10kg/22lb-200kg/440lb, please follow as below steps:

1. Place the scale on a flat, hard surface. Avoid any carpeting.
2. Turn on the scale by stepping of and off, waiting for "0.0" display on the screen.
3. Turn on Vitafit app. Step on the scale with bare feet and stand still until "0000" appears on the display, it indicates the scale is detecting the body fat. Once body fat detection is complete, you can view your data of BMI, body fat, body water, muscle mass and protein on the scale. The scale first display BF, Weight and BW, after that display MM, BMI and PR. Meantime all 13 of composition data will also display on the app.

If the weight is under 10kg/22lb or over 200kg/400lb, please repeat above steps, but the scale only has weight function, the scale and app will display weight and BMI result.

Connecting with Other Fitness Apps

Google Fit

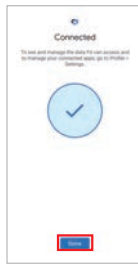
Authorization Steps on the Vitafit App

1. On the page of My Account, click "Health Connect".
2. Please Install Health Connect if the device system below Android 14, and then start authorize.
- If the system is Android 14 and above, please skip the install step, just directly click "Authorize Immediately".
3. click all allow button, then the authorization is successful.




Connect steps on Google Fit

1. Open the Google Fit, and click "Get started".
2. Click "Set up".
3. Click "Done" to finish connected.
4. After successfully connected, your measurement data will sync with Google Fit automatically.




Samsung Health

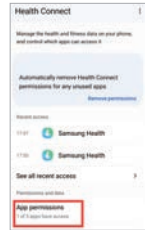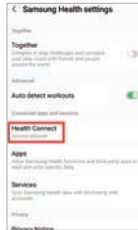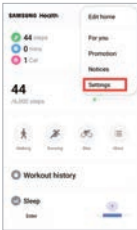1. Firstly, please make same steps of authorization on the Vitafit app as above Google Fit mentioned.
2. After Vitafit app connected with Health Connect, please start Settings on Samsung Health as follow steps.
 - 2.1. Open the Samsung Health, click "Settings"
 - 2.2. Click "Health Connet"
 - 2.3. Click "App permissions"

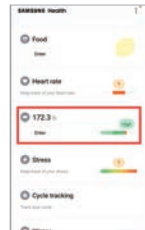
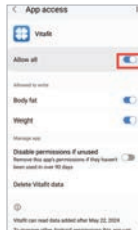
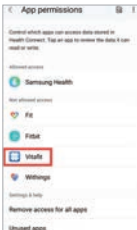
- 2.4. Click "Vitafit" to allow access
- 2.5. Click "Allow all" button to finish connecting
- 2.6. After successfully connected, your measurement data will sync with Samsung Health automatically



Apple Health

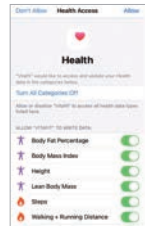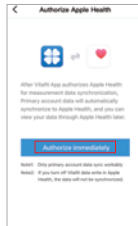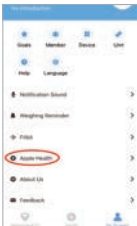1. On the page of My Account, Select Apple Health from the menu.
 2. Click Authorize and the App will authorize immediately.
 3. Open the Health App on your iPhone, then tap Sources. Be sure to turn on your Bluetooth®.
 4. Select Vitafit from the list.
 5. Turn on all categories to allow the Vitafit app to work.
- Note: In IOS system, couldn't be allowed to Sync with Google Fit.


Fitbit

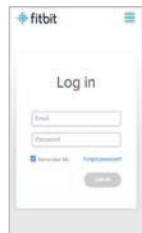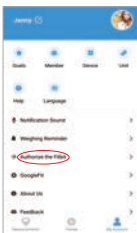1. On the page of My Account, Select Fitbit from the menu.
2. Turn on Fitbit, then sign into your Fitbit account.

3. Tap Allow to access the Fitbit app.

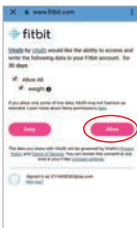
4. After successfully connecting, your data will sync to the Fitbit app. You will be able to view your progress both on the app as well as online.

Troubleshooting

The scale doesn't turn on.

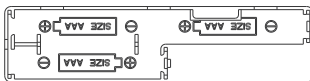If the scale won't turn on or the battery is low, the batteries need to be replaced, and make sure the battery is installed properly.

When I stand on the scale, it doesn't respond.

You may need to restart the scale. Remove the batteries and reinsert them to restart the scale.

The scale doesn't connect to the Vitafit app.

- Make sure your phone's operating system is running on IOS9.0+ or Android 4.3+
- Close and relaunch the Vitafit app. Make sure you're using the latest version.
- Make sure your phone has Bluetooth turned on. For Android users, make sure your Location is turned on as well.
- Make sure the scale is not currently connected to any other phone or app.
- The scale should only be connected to your phone via the Vitafit app. Do not connect the scale to your phone's Bluetooth settings.

The scale readings seem unstable or inaccurate.

- Make sure the scale is on a hard, flat surface.
- Make sure the scale is calibrated before use, to calibrate the scale, 1: Step on the scale with a single foot to activate it and then step off; 2: The display will show "CAL" and then "0.0" or "0.00" to indicate calibrating is complete.

My scale will only measure my weight, not other biometrics.

- If you're wearing shoes or socks, the scale cannot measure biometrics such as body fat. Step on the scale with bare feet.
- Make sure to place your bare feet on the scale's conductive areas.
- Depending on which biometrics you want to see, make sure the scale is connected to the Vitafit app
- Check to see if you are in which Mode. In Weight Only Mode, the scale will only provide the weight and BMI.

Package Contents

Serial Number	Item	Quantity
1	Smart Body Fat Scale	1
2	1.5V AAA Battery	3
3	User Manual	1

Specifications

Capacity: Min 5 kg/11lb/0:11lb
Max 250kg/550lb/39st

Notes: When weight range between from 10kg/22lb-200kg/440lb, body composition such as body fat could be scientifically analyzed; When weight range LESS THAN 10kg/22lb OR MORE THAN 200kg/440lb, ONLY WEIGHT COULD BE MEASURED.

Weight unit: kg/lb/st

Graduation: 0.05kg/0.1lb/0.1lb

Dimensions: 305x305x25.7mm

Display: LCD

Platform: Tempered glass

Batteries: 3xAAA

Warranty Information

Homebuds guarantees all products to be of the highest quality in material and craftsmanship, and provides all products with 3 years warranty effective from the date of purchase.

For more information please visit www.homebuds.com

Email: service@homebuds.com

FCC COMPLIANCE STATEMENT

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Warning: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: -Reorient or relocate the receiving antenna -Increase the separation between the equipment and receiver. -Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. -Consult the dealer or an experienced radio/TV technician for help.

ISED COMPLIANCE STATEMENT

This device contains licence-exempt transmitters that comply with innovation, Science and Economic Development Canada's licence-exempt RSS(s).

Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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Notice:

This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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**EU Declaration of Conformity
for
RED Directive 2014/53/EU**

We,
Shenzhen Shine Industrial Co., Ltd.

hereby, declare that the essential requirements set out in the RED Directive 2014/53/EU have been fully fulfilled on our product with indication below:

Product Name: Smart Body Fat Scale

Brand Name: /

Model No.: VT715, VT7151, VT7152, VT7153, VT7155, VT7156, VT7158, VT7159, HB912, HB9121, HB9122, HB9123, HB9125, HB9126, HB9127, HB9128, HB9129

The following standards have been applied for the investigation of compliance:

ETSI EN 300 328 V2.2.2 (2019-07)

EN 50663:2017

EN 62479:2010

ETSI EN 301 489-1 V2.2.3 (2019-11)

ETSI EN 301 489-17 V3.3.1 (2024-09)

EN IEC 55014-1:2021

EN IEC 55014-2:2021

EN IEC 61000-3-2:2019+A1:2021+A2:2024

EN 61000-3-3:2013+A1:2019+A2:2021

EN 60335-1:2012+A11:2014+A13:2017+

A1:2019+A14:2019+A2:2019+A15:2021+A16:2023

EN 62233:2008

And apply notified body assessment:

Notified Body number 1177

Timco Engineering, Inc.

13146 NW 86th Drive Suite 400 Alachua, FL 32615,
America

Software Version:	V4
Hardware Version:	V3
System Components	
Component 1	/
Optional Components	
Optional 1	/

Furthermore, the ISO requirement for the in-process quality control procedure as well as the manufacturing process has been reached. The technical document as well as the test reports will be kept for a period at least 10 years after the last product has been manufactured at the disposal of the relevant national authorities of any Member State for inspection.

Detail contact information for this declaration has been listed below as the window of any issues relevant for this declaration.

Manufacturer Contact

Company: Shenzhen Shine Industrial Co., Ltd.

Address: 2-3/F, Bldg 5, 1st Industrial Zone, Changzhen Community, Yutang town, Guangming District, Shenzhen, Guangdong, China

Name & Title: Xinqiang Xu / General Manager

Tel No.: +86-755-2717-5093

Fax No.: +86-755-2717-5261

E-mail: ricky@iascent.com.cn

Date: 4/10/2025

Caution:

Use the Product in the environment with the temperature Between -10°C and 50°C ; Otherwise, it may damage your product. Products can only be used below 2000m altitude

For the following equipment:

Product Name: Smart Body Fat Scale

Brand Name: /

Model No.: VT715, VT7151, VT7152, VT7153, VT7155, VT7156, VT7158, VT7159, HB912, HB9121, HB9122, HB9123, HB9125, HB9126, HB9127, HB9128, HB9129

Shenzhen Shine Industrial Co., Ltd.

E-mail: ricky@iascent.com.cn

hereby declares that this [Name: Bathroom Scale, Model: VT715, VT7151, VT7152, VT7153, VT7155, VT7156, VT7158, VT7159, HB912, HB9121, HB9122, HB9123, HB9125, HB9126, HB9127, HB9128, HB9129] is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

CAUTION
RISK OF EXPLOSION IF BATTERY IS REPLACED
BY AN INCORRECT TYPE.
DISPOSE OF USED BATTERIES ACCORDING
TO THE INSTRUCTIONS

This product is intended for sale and application in a business environment.

RED Article 10 2

-This product can be used across EU member states

RED Article 10 10

-The product is class 1 product, No restrictions

The RF distance between body and product is 5M

Bluetooth Version:Bluetooth V5.0(BLE Mode)

Frequency Range:2402MHz-2480MHz

Max.RF Output Power:6.76dBm (EIRP)

Homebuds

Inspired by Life

Factory: SHENZHEN SHINE INDUSTRIAL CO., LTD.
Address: 2&3F, Bldg5, 1st Industrial Zone, Changzhen Community,
Yutang Street, Guangming District, Shenzhen City, Guangdong
Province, China, 518132
E-mail: service@homebuds.com

Company: Apex CE Specialists GmbH
Address: Grafenberger Allee 277, 40237 Düsseldorf, Westfalen, Deutschland
Zip Code: 40237
Contact: Wells Yan
E-mail: info@apex-ce.com
Tel: +491776995991

Company: APEX CE SPECIALISTS LIMITED
Address: 6f East Union Street, Manchester, England, M16 9AE
Contact: Wells
E-mail: Info@apex-ce.com
Tel: +441616371080